



Setup Guide for Ascend Imaging (Windows version only).


- 1) Install the REALCLOUD HD1 camera button software utility. The installer is located on the REALCLOUD HD1 flash drive in the Button Software folder. Or REALCLOUD Website <http://www.realcloudimaging.com/support/>
Scroll down the Support Page to the Downloads Section. Click the “Capture Button Software” button. This will start downloading the Button-Tray-Utility.zip , once downloaded open the zip file and Install the REALCLOUD HD1 camera button tray utility software.
- 2) Select the Next button on the Welcome screen.

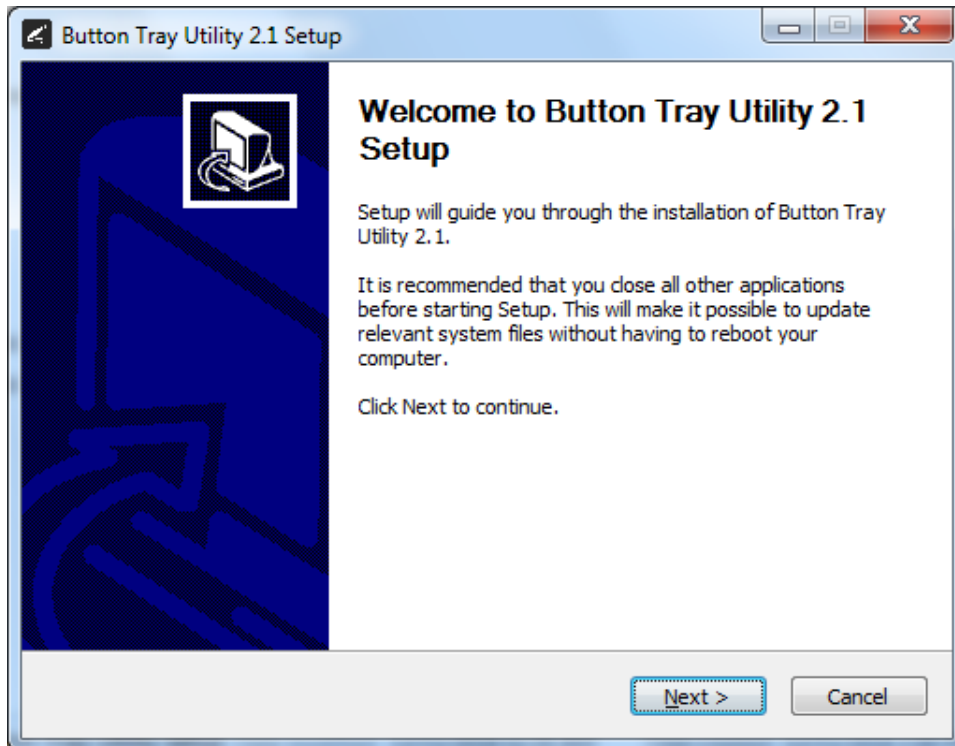


3) Select the install destination folder and click on the Install button.

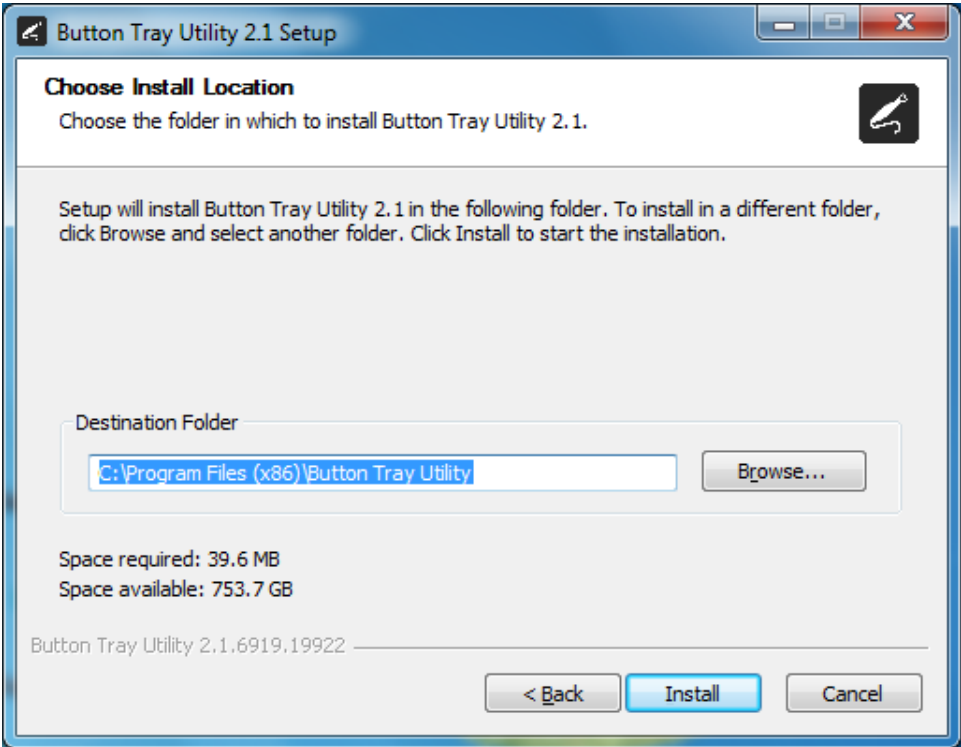


4) Plug the REALCLOUD HD1 camera into the computer and click on the OK button to start the installation process.

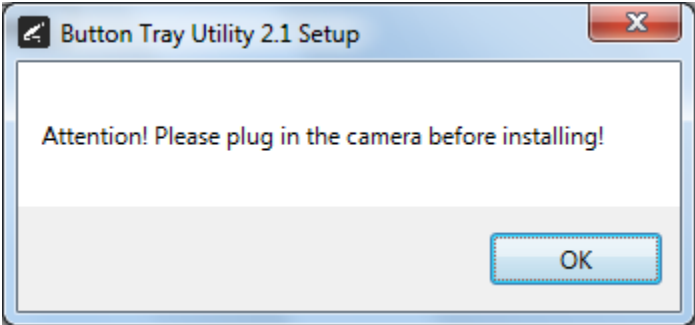


5) Click on the Next button to continue.

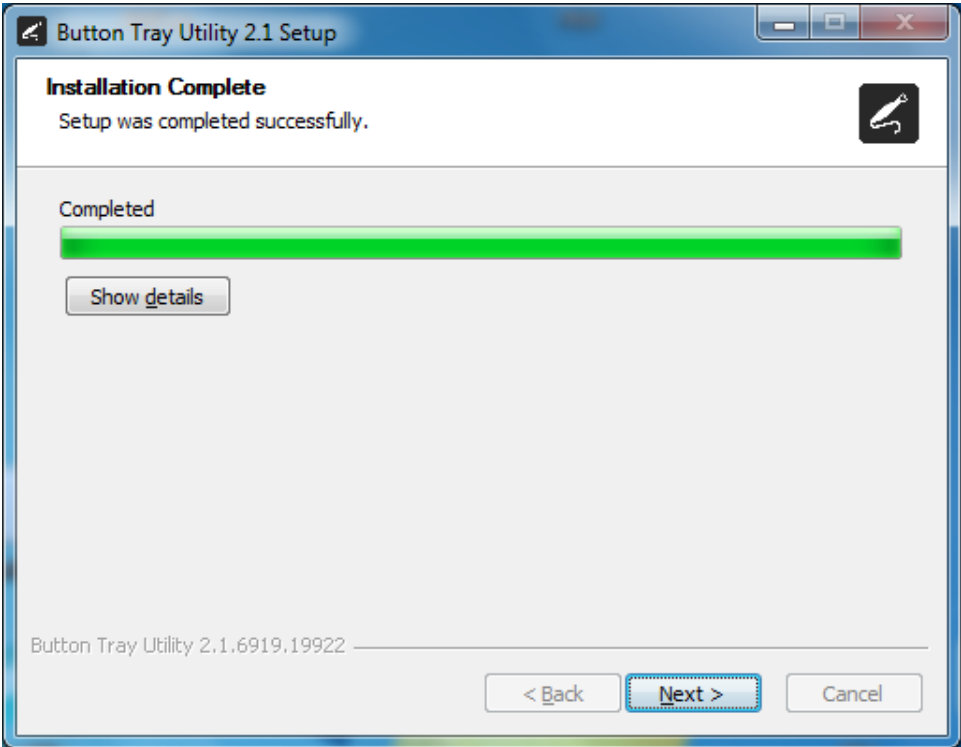

6) Select a reboot option (if prompted) and click on the Finish button.

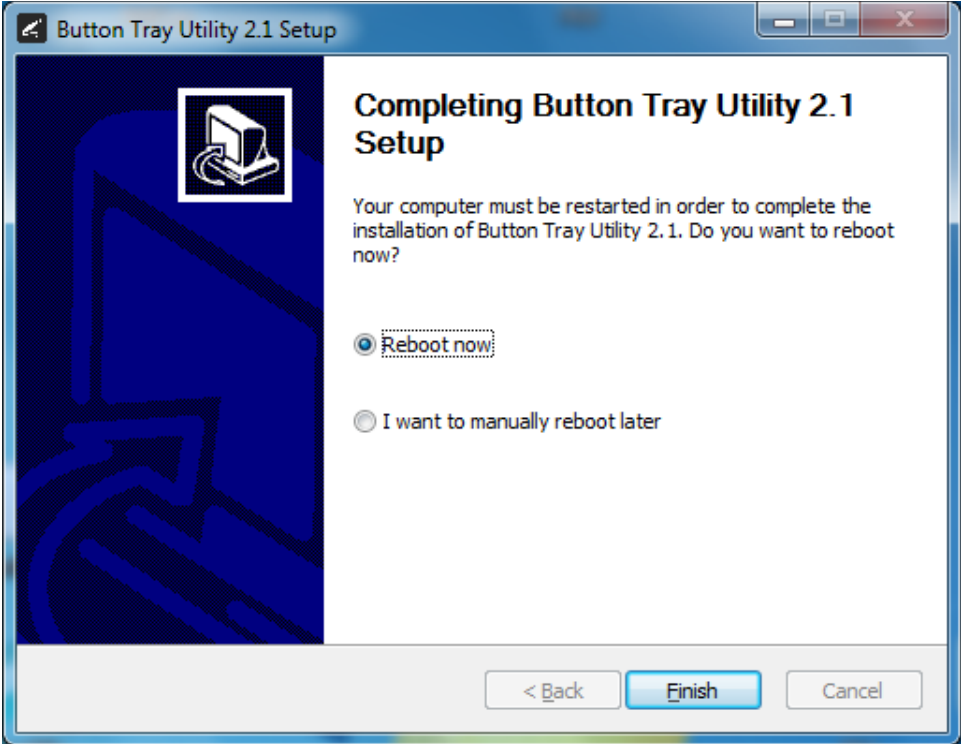

7) Right click on the Button Tray Utility icon in your system tray and select Preferences.

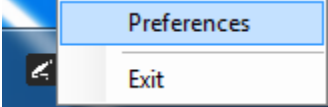

- 8) Click In the first text box for Short Press Keys and press the “s” key one time on your keyboard and leave the second text box blank.
- 9) Click In the text box for Long Press Keys and press the “x” key one time on your keyboard, then click on the Done button.

The image shows a 'Preferences' dialog box with a close button (X) in the top right corner. It contains the following elements:


- Short Press Keys:** Two text input boxes. The first box contains the letter 's', and the second box is empty.
- Long Press Keys:** A single text input box containing the letter 'x'.
- Status:** A section containing three checkboxes:
 - Camera Registered
 - Camera Streaming
 - Button Click
- Done:** A button at the bottom center of the dialog box.

10) Open the Dentrix Ascend - Imaging web page, go to Acquire Images and select the 2D Oral/Facial Photo Images.

Imaging Procedures	
Intraoral Full Mouth Images	D0210
Intraoral Periapical Image	D0220
Intraoral Occlusal Image	D0240
Extraoral Posterior Image	D0251
Bitewing Single Image	D0270
Bitewing Two Images	D0272
Bitewing Four Images	D0274
Bitewing Seven Images	D0277
2 Bitewing + 2 Occlusal Images	2BW2OCC
2 Bitewing + 2 Periapical Images	2BW2PA
4 Bitewing + 2 Periapical Images	4BW2PA
4 Bitewing + 4 Periapical Images	4BW4PA
Panoramic Image	D0330
Cephalometric Image	D0340
2D Oral/Facial Photo Images	D0350

- 11) Set the “Acquisition method for this imaging exam” to “RealCloud HD1 1080P” and click on the Proceed with Acquisition button.

Task to Perform

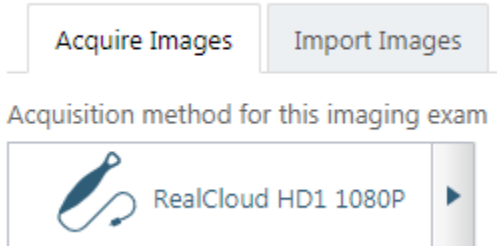

- 12) A short/quick camera button press, and release will pause the live image video from the REALCLOUD HD1 camera. When a video image is paused, a short/quick camera button press, and release will save the image and return to live video. When a video image is paused, a long camera button press, hold (2 second), and release will return to live video not saving the paused image.