



## Setup Guide for Carestream/Kodak Dental Imaging Software 6.12 and higher.


- 1) Install the REALCLOUD HD1 camera button software utility. The installer is located on the REALCLOUD HD1 flash drive in the Button Software folder. Or REALCLOUD Website <http://www.realcloudimaging.com/support/>  
Scroll down the Support Page to the Downloads Section. Click the “Capture Button Software” button. This will start downloading the Button-Tray-Utility.zip , once downloaded open the zip file and Install the REALCLOUD HD1 camera button tray utility software.
- 2) Select the Next button on the Welcome screen.

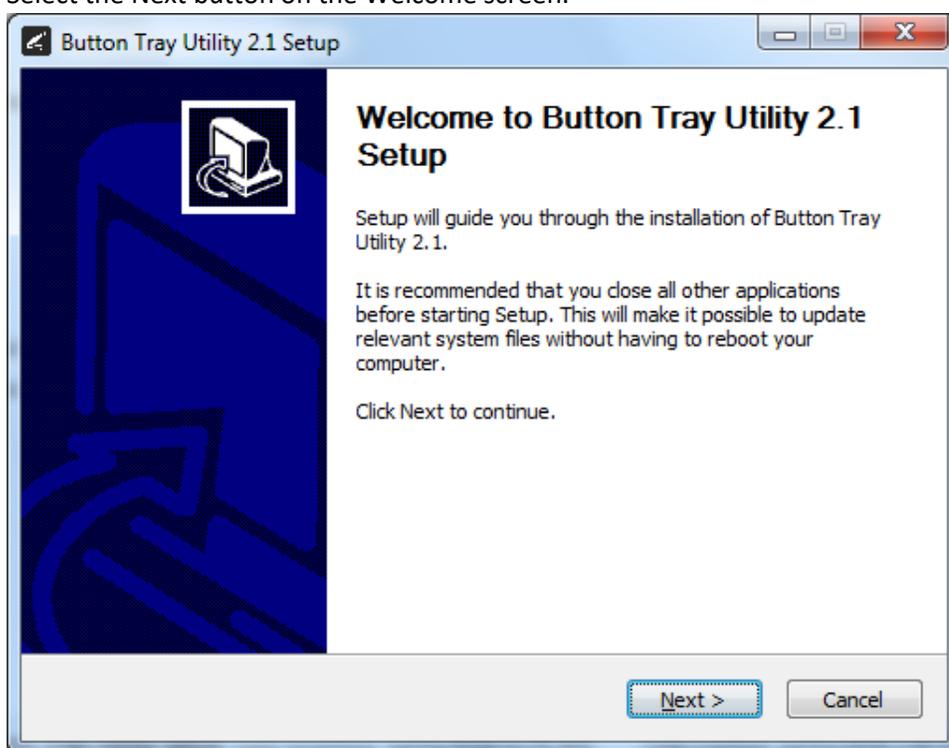


- 3) Select the install destination folder and click on the Install button.

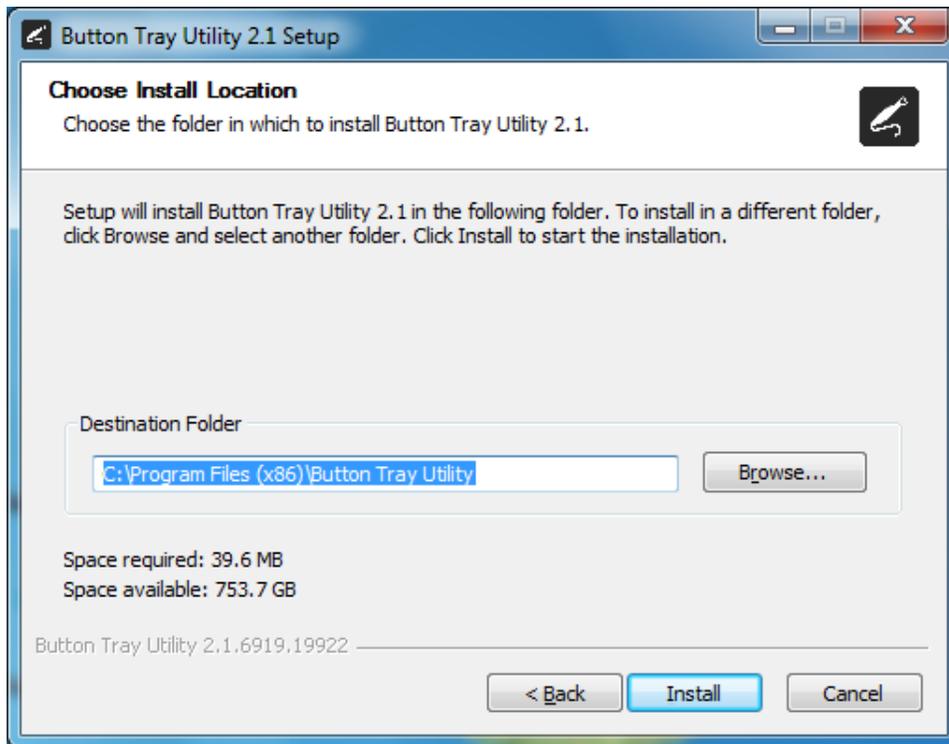


- 4) Plug the REALCLOUD HD1 camera into the computer and click on the OK button to start the installation process.

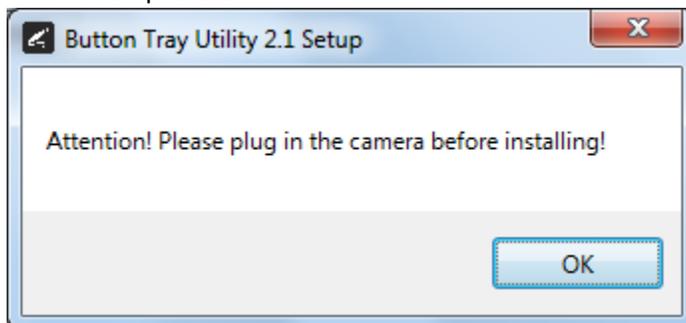


- 5) Click on the Next button to continue.

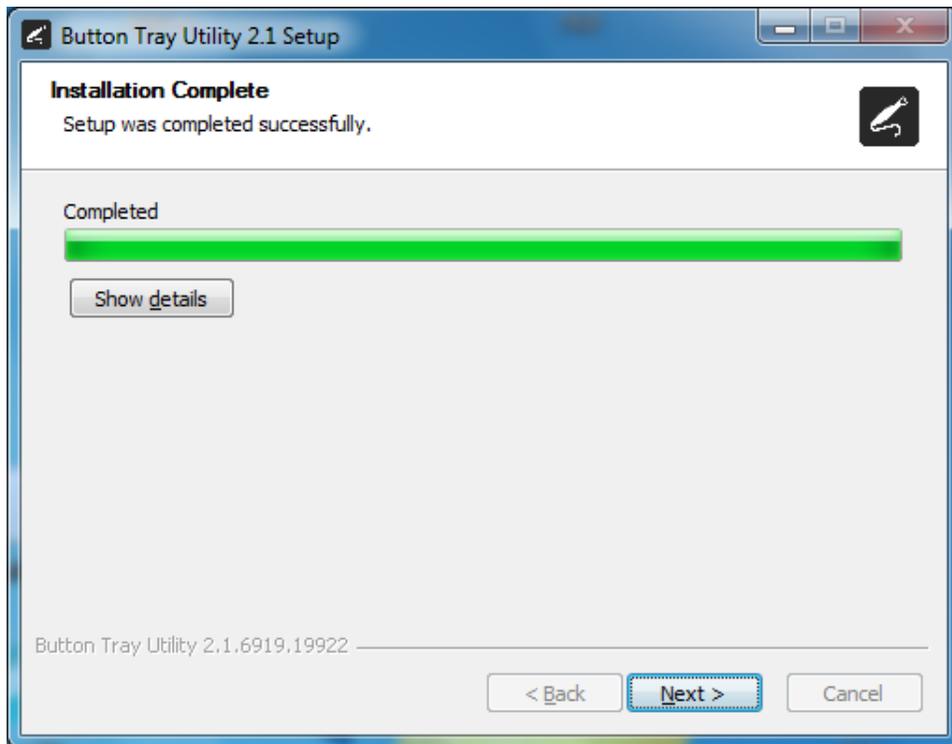


- 6) Select a reboot option (if prompted) and click on the Finish button.

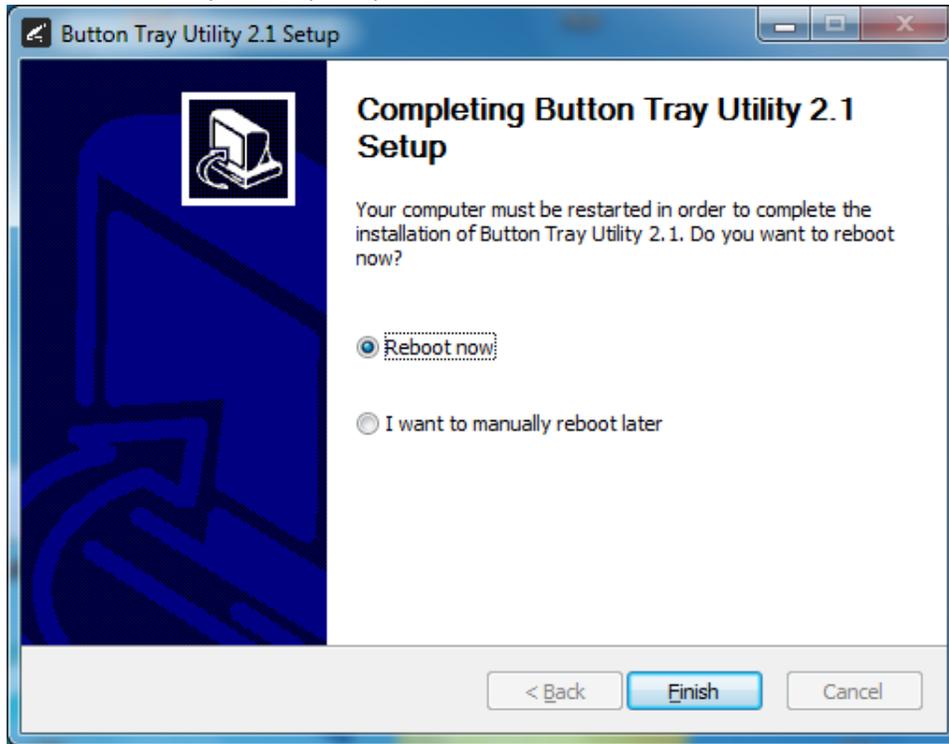


- 7) Right click on the Button Tray Utility icon in your system tray and select Preferences.

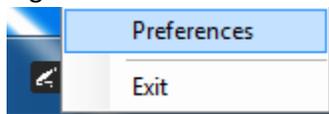

- 8) Click In the first text box for Short Press Keys and type in "{F5}" on your keyboard and leave the second text box for Short Press Keys blank.
- 9) Leave the text box for Long Press Keys blank, then click on the Done button.

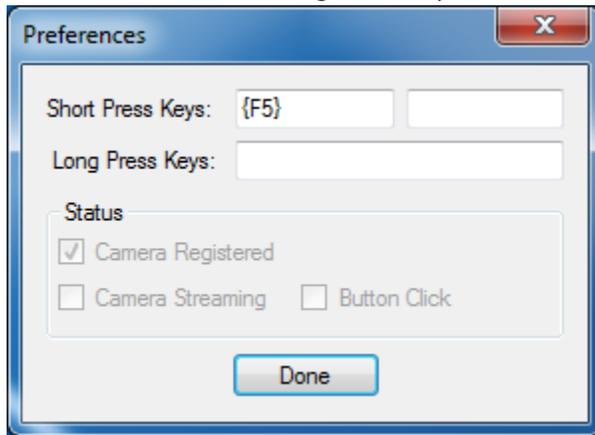

- 10) Open Carestream/Kodak Dental Imaging Software application, go to Options and select the Preferences. On the camera tab, be sure the "Use a video capture source" box is checked. If not, check the box, then exit the Carestream/Kodak Dental Imaging software.

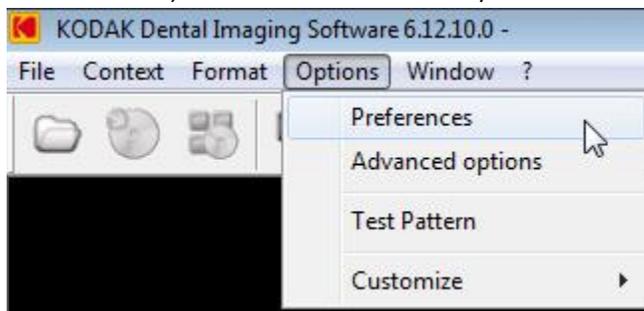

- 11) In the Carestream/Kodak Dental Imaging Software with a patient selected, click on the New intraoral camera image button to begin acquiring images.




NOTE: If you are unable to open the video window in Carestream/Kodak Dental Imaging software, please verify your Kodak Dental Imaging software license includes this feature and if it doesn't contact Carestream/Kodak for info on enabling the video capture module.

- 12) If video from the REALCLOUD HD1 is not displayed, click on the gear icon on the right side of the video window.



- 13) Select RCI 1080P HD from the dropdown list of installed video devices and click on the OK button.

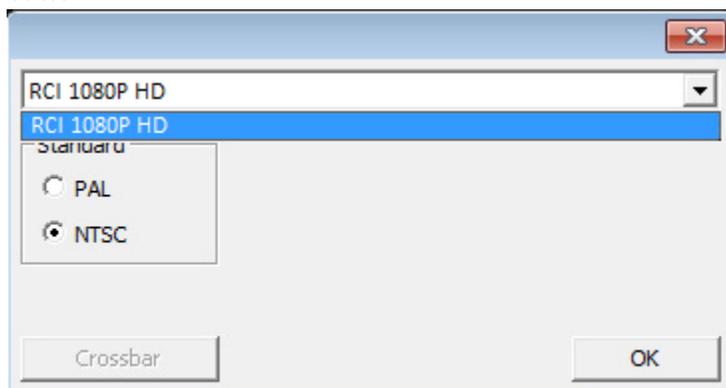

- 14) If the image color is not displayed correctly, click on the default button to the right of the video settings sliders.



- 15) A short/quick camera button press, and release will pause the live video from the REALCLOUD HD1 camera. When a video image is paused, a second short/quick camera button press, and release will stage an image to be saved will return to live video.