



## Setup Guide for Dexis Imaging Suite 10.x and higher.


- 1) Install the REALCLOUD HD1 camera button software utility. The installer is located on the REALCLOUD HD1 flash drive in the Button Software folder. Or REALCLOUD Website <http://www.realcloudimaging.com/support/>  
Scroll down the Support Page to the Downloads Section. Click the “Capture Button Software” button. This will start downloading the Button-Tray-Utility.zip , once downloaded open the zip file and Install the REALCLOUD HD1 camera button tray utility software.
- 2) Select the Next button on the Welcome screen.

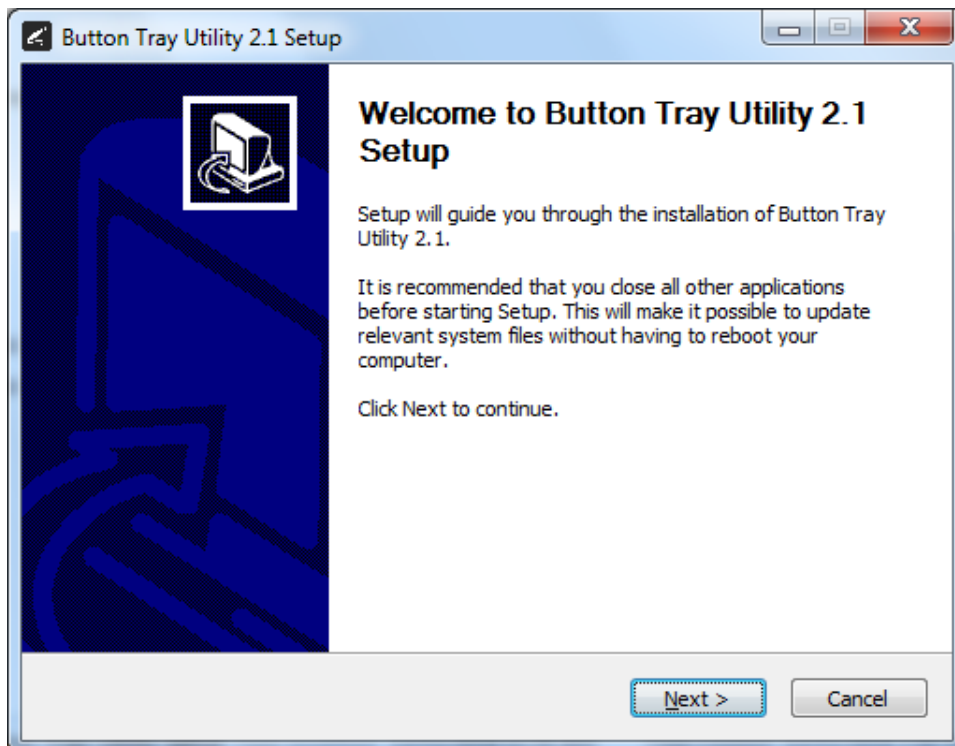


- 3) Select the install destination folder and click on the Install button.

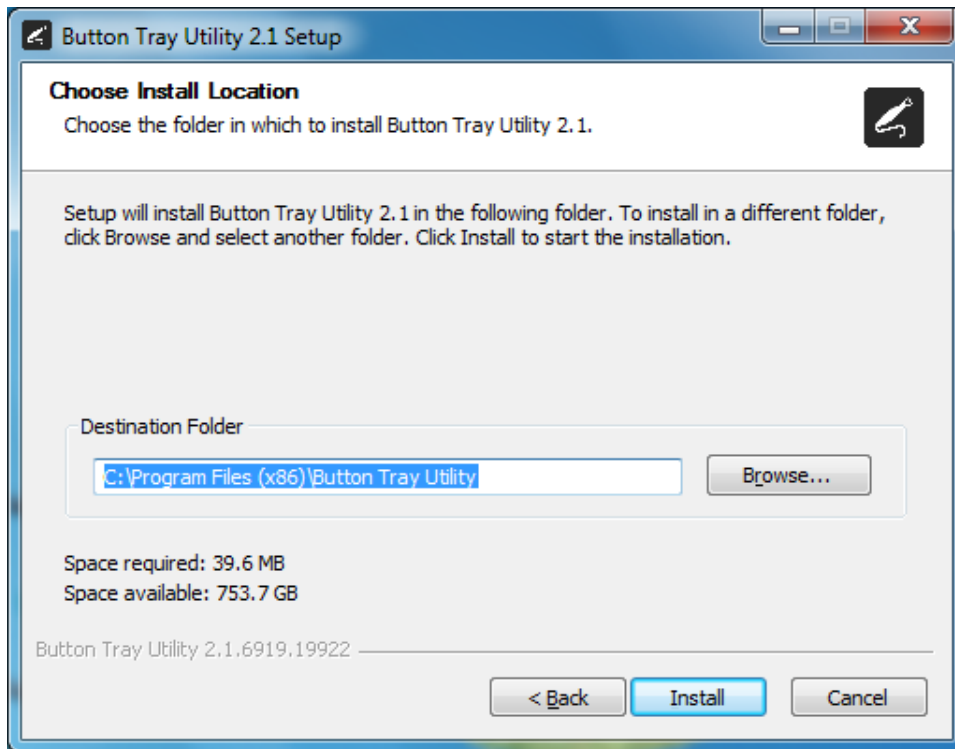


- 4) Plug the REALCLOUD HD1 camera into the computer and click on the OK button to start the installation process.

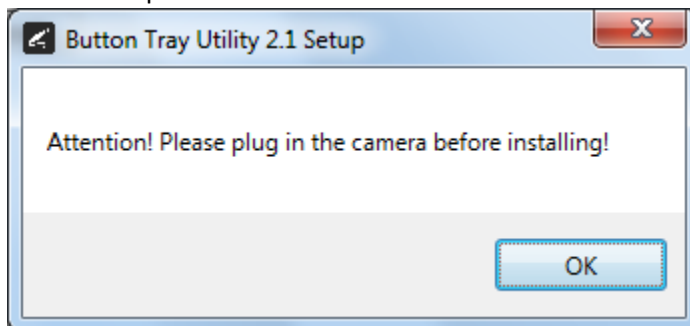


- 5) Click on the Next button to continue.

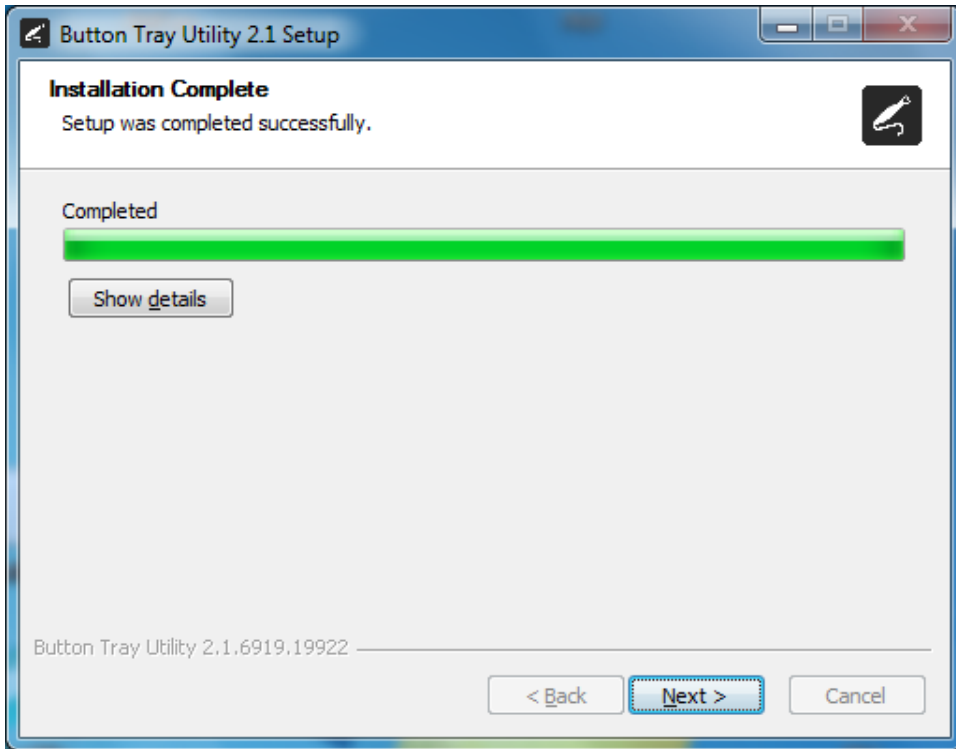


- 6) Select a reboot option (if prompted) and click on the Finish button.

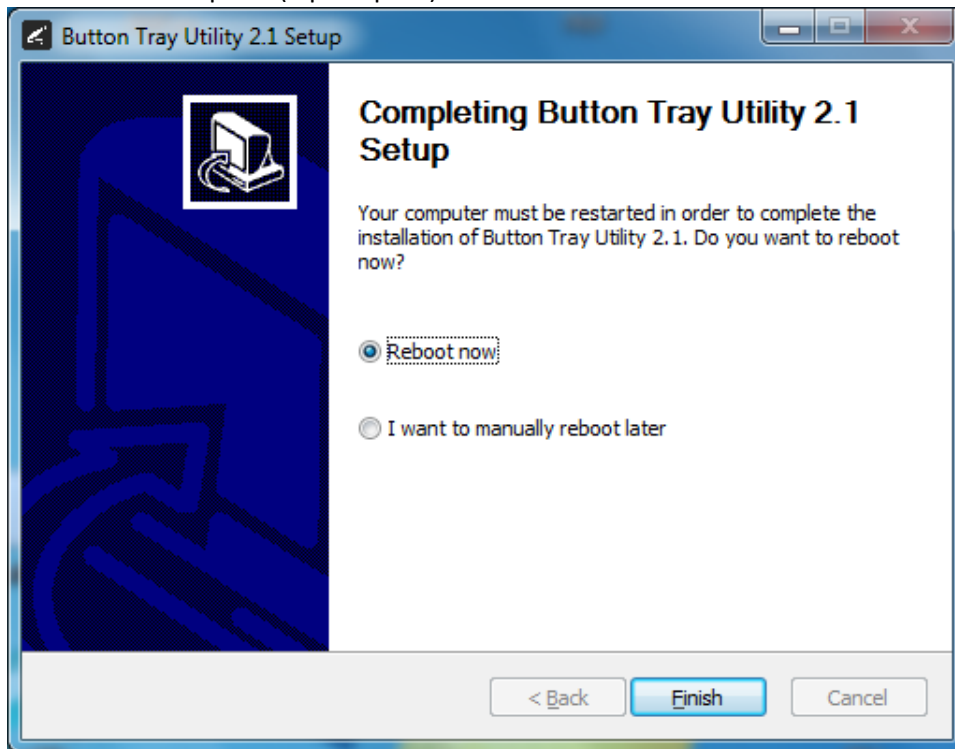


- 7) Right click on the Button Tray Utility icon in your system tray and select Preferences.

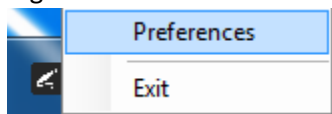


- 8) Click In the first text box for Short Press Keys and press the “spacebar” key one time on your keyboard and in the second text box for Short Press Keys press the “t” key one time on your keyboard and the “spacebar” key one time on your keyboard.
- 9) Click In the text box for Long Press Keys and press the “spacebar” key one time on your keyboard, then click on the Done button.

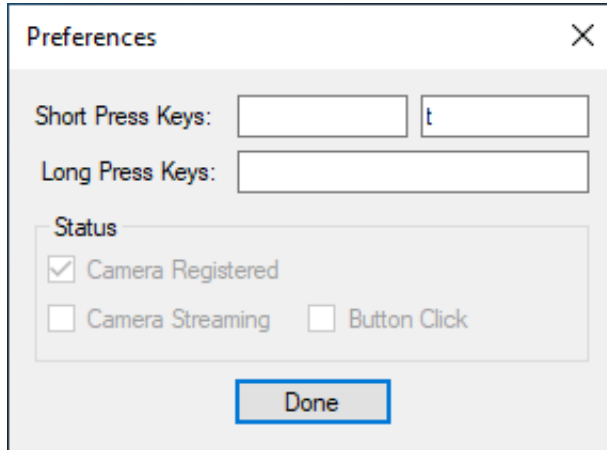

- 10) Open DEXIS Imaging Suite application, go to Preferences and select the Video Acquisition page.

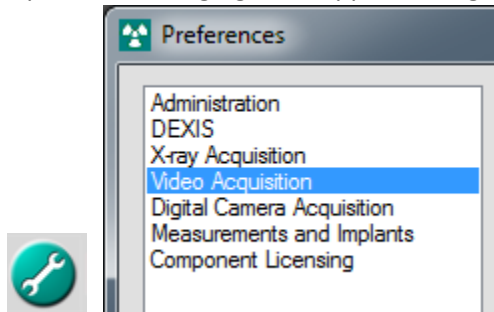

- 11) Set the Primary Device to “Any Device” and make sure the “Use VMR9 Rendering”, “Ignore Still Pin” and “Use Capture Pin for Preview” options are selected and click on the Apply button followed by the OK button.

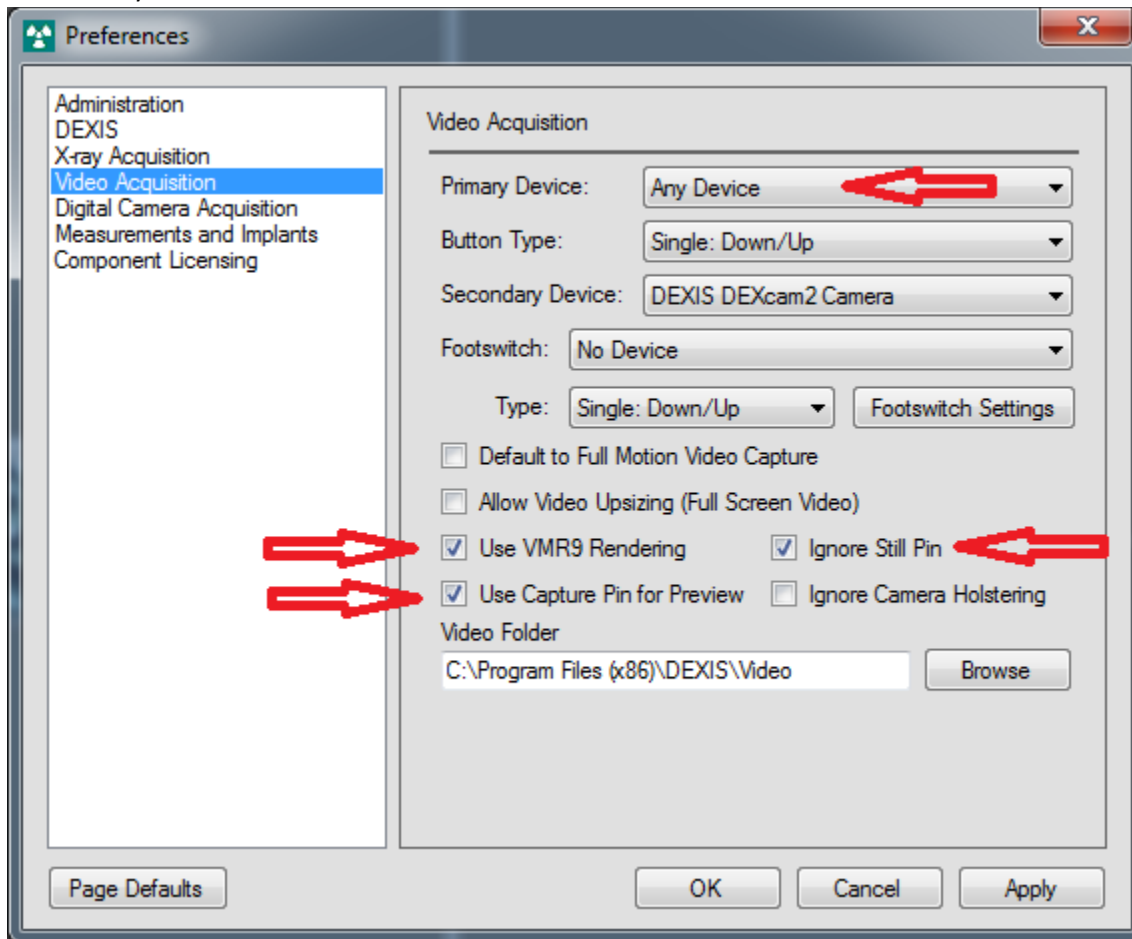

- 12) Select intra-oral camera mode or extra-oral camera mode icons and then select the DEXvideo button to begin acquiring images.



- 13) A short/quick camera button press, and release will pause the live video from the REALCLOUD HD1 camera. When a video image is paused, a second short/quick camera button press, and release will stage an image to be saved when DEXvideo is closed and will return to live video. When a video image is paused, a long camera button press, hold (2 seconds), and release will return to live video without saving the image.