


## Setup Guide for GoodDrs Viewer 2 v2.3.3 and higher


1. Install the REALCLOUD HD1 camera button software utility. The installer is located on the REALCLOUD HD1 flash drive in the Button Software folder. Or REALCLOUD Website <http://www.realcloudimaging.com/support/>

Scroll down the Support Page to the Downloads Section. Click the “Capture Button Software” button. This will start downloading the Button-Tray-Utility.zip , once downloaded open the zip file and Install the REALCLOUD HD1 camera button tray utility software.


2. Select the Next button on the Welcome screen.

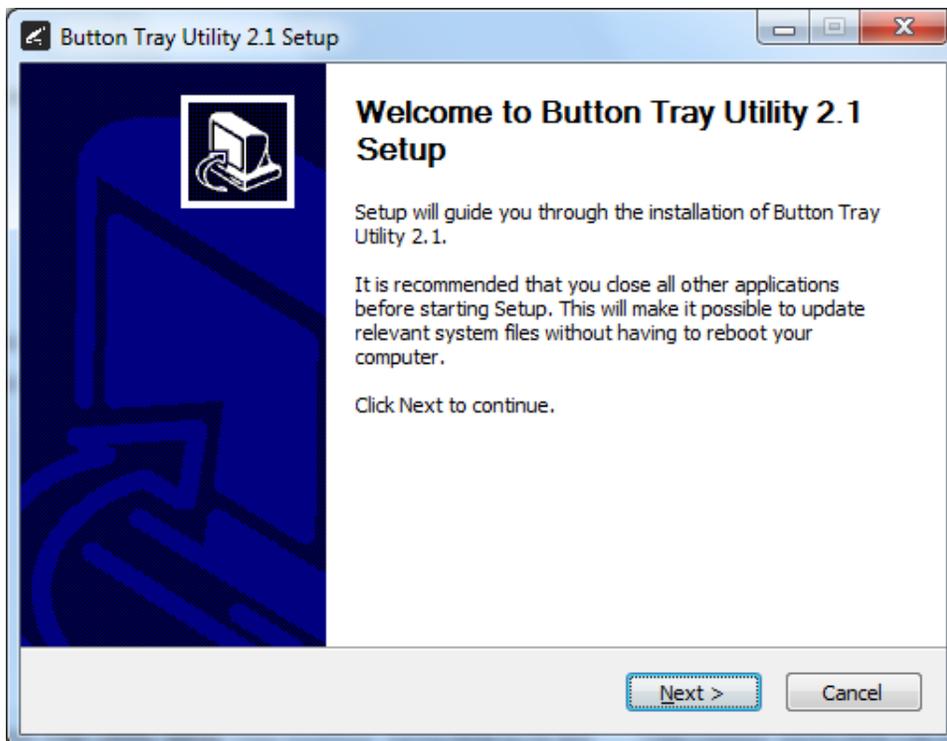


3. Select the install destination folder and click on the Install button.

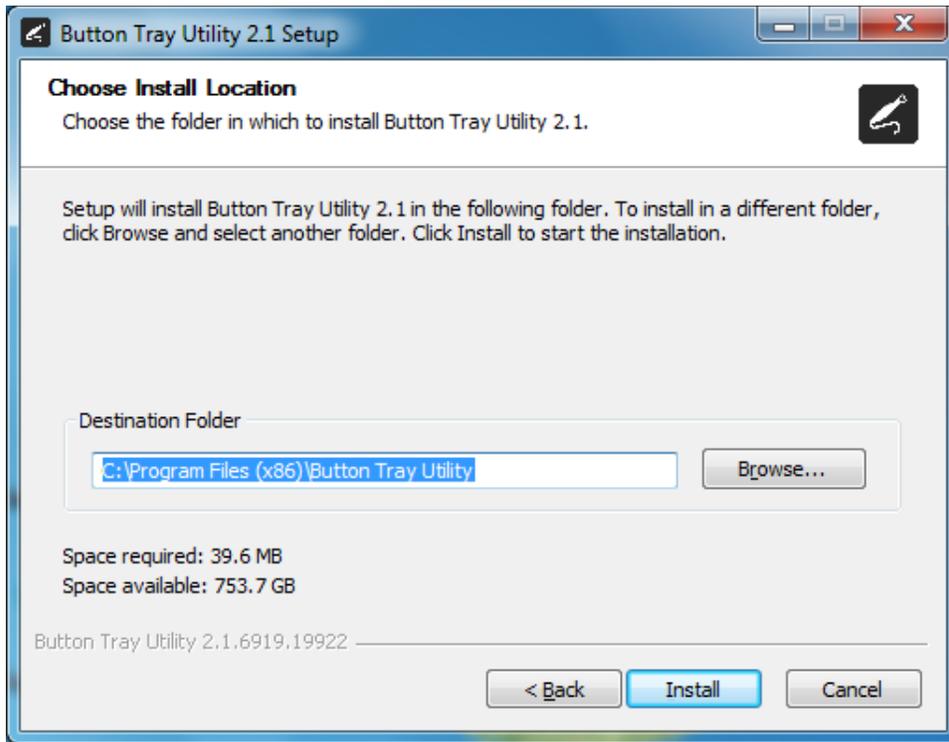


4. Plug the REALCLOUD HD1 camera into the computer and click on the OK button to start the installation process.

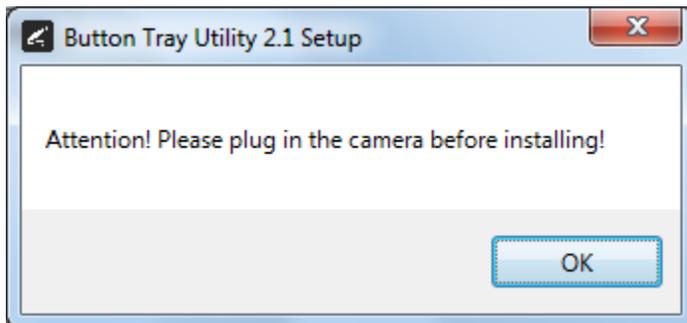


5. Click on the Next button to continue.

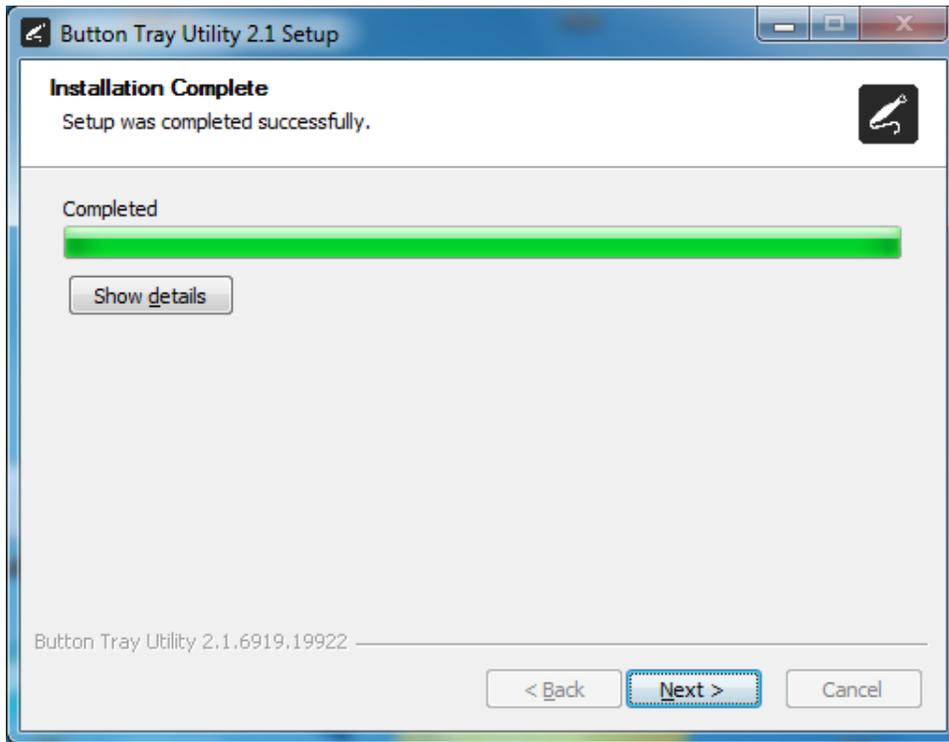

6. Select a reboot option (if prompted) and click on the Finish button.

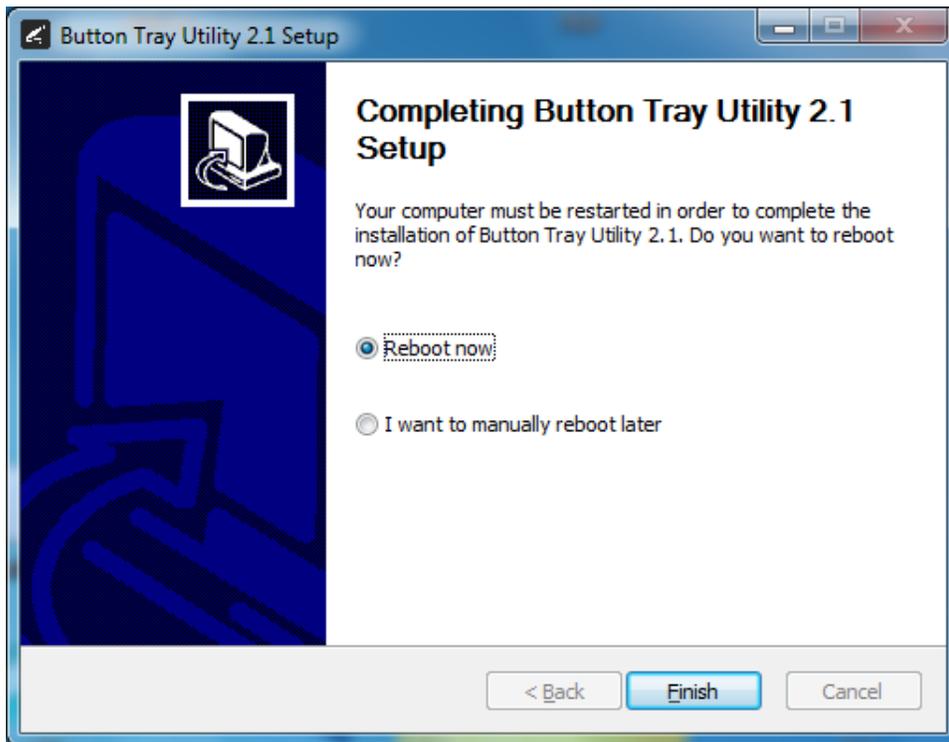


7. Right click on the Button Tray Utility icon in your system tray and select Preferences.

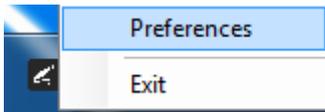


8. Click In the first text box for Short Press Keys and press the “spacebar” key one time on your keyboard.


**Note:** The second text box for Short Press Key and the text box for Long Press Keys are left blank.

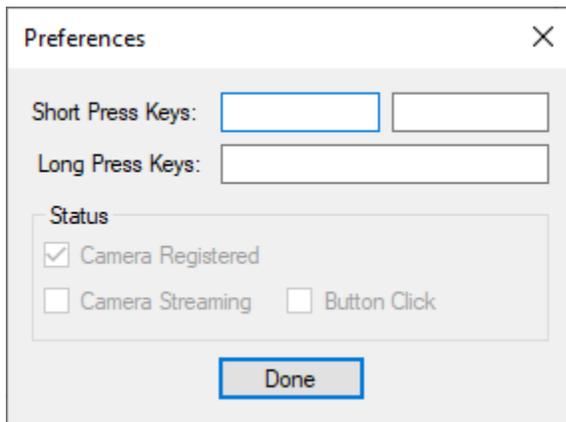


9. Open File Explorer and go to C:\ProgramData\GoodDrs

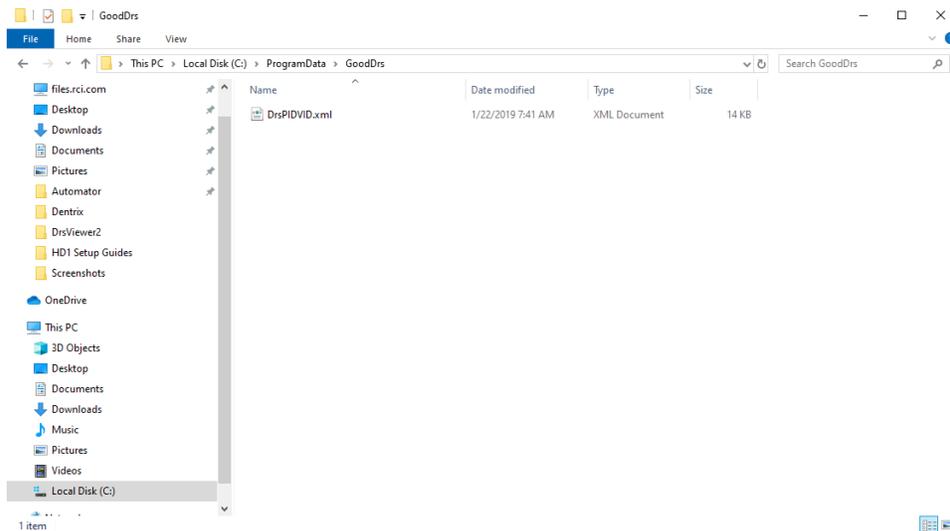

10. Right-click and Edit the DrsPIDVID.xml file

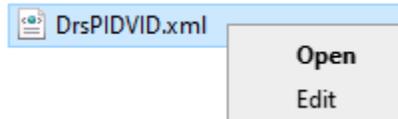


11. On line 5 change from VID\_2560\_PID\_D051 to VID\_259C\_PID\_6C01

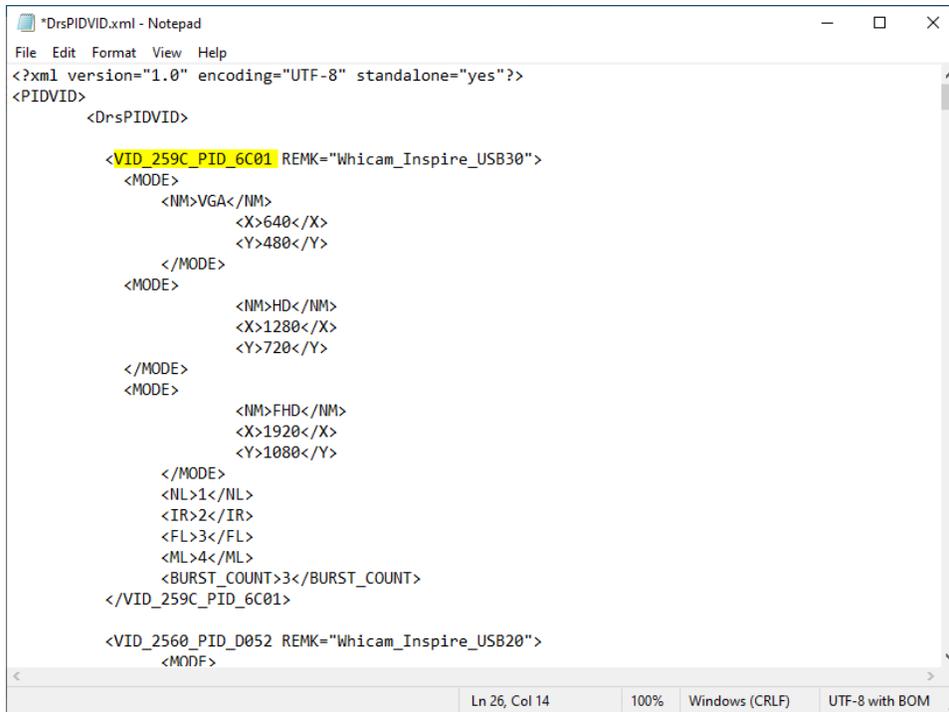A screenshot of a Notepad window titled '\*DrsPIDVID.xml - Notepad'. The window contains XML code. The line '<VID\_259C\_PID\_6C01 REMK="Whicam\_Inspire\_USB30">' is highlighted in yellow. The code is as follows:

```
<?xml version="1.0" encoding="UTF-8" standalone="yes"?>
<PIDVID>
  <DrsPIDVID>
    <VID_259C_PID_6C01 REMK="Whicam_Inspire_USB30">
      <MODE>
        <NM>VGA</NM>
        <X>640</X>
        <Y>480</Y>
      </MODE>
      <MODE>
        <NM>HD</NM>
        <X>1280</X>
        <Y>720</Y>
      </MODE>
      <MODE>
        <NM>FHD</NM>
        <X>1920</X>
        <Y>1080</Y>
      </MODE>
      <NL>1</NL>
      <IR>2</IR>
      <FL>3</FL>
      <ML>4</ML>
      <BURST_COUNT>3</BURST_COUNT>
    </VID_259C_PID_6C01>
    <VID_2560_PID_D052 REMK="Whicam_Inspire_USB20">
      <MODE>
```

The status bar at the bottom shows 'Ln 26, Col 14', '100%', 'Windows (CRLF)', and 'UTF-8 with BOM'.


## 12. On line 26 change from VID\_2560\_PID\_D051 to VID\_259C\_PID\_6C01

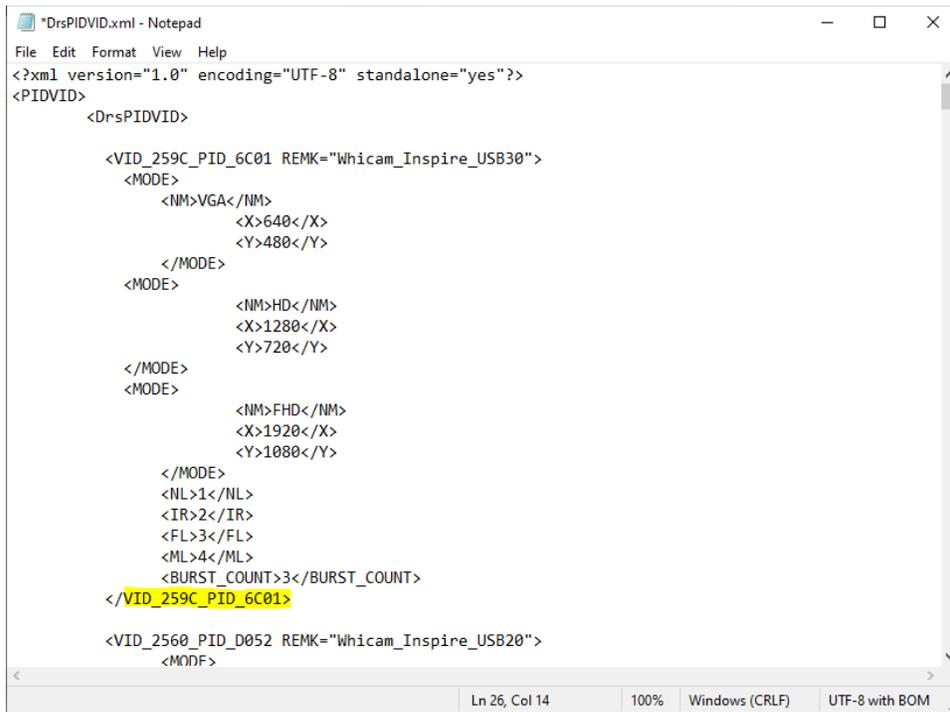



```
*DrsPIDVID.xml - Notepad
File Edit Format View Help
<?xml version="1.0" encoding="UTF-8" standalone="yes"?>
<PIDVID>
  <DrsPIDVID>
    <VID_259C_PID_6C01 REMK="Whicam_Inspire_USB30">
      <MODE>
        <NM>VGA</NM>
        <X>640</X>
        <Y>480</Y>
      </MODE>
      <MODE>
        <NM>HD</NM>
        <X>1280</X>
        <Y>720</Y>
      </MODE>
      <MODE>
        <NM>FHD</NM>
        <X>1920</X>
        <Y>1080</Y>
      </MODE>
      <NL>1</NL>
      <IR>2</IR>
      <FL>3</FL>
      <ML>4</ML>
      <BURST_COUNT>3</BURST_COUNT>
    </VID_259C_PID_6C01>
    <VID_2560_PID_D052 REMK="Whicam_Inspire_USB20">
      <MODE>

```



## 13. Save the file to the Desktop

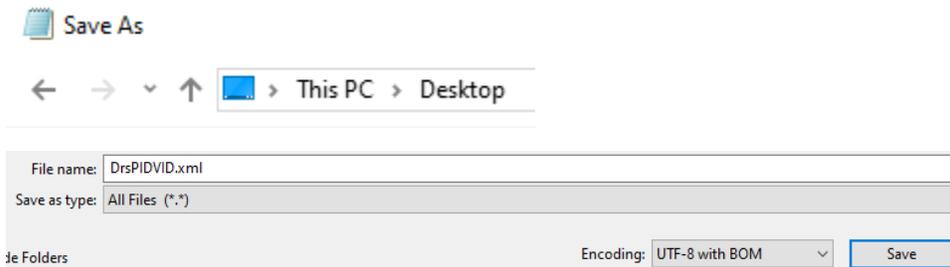

14. Copy the file saved to the Desktop to the C:\ProgramData\GoodDrs folder

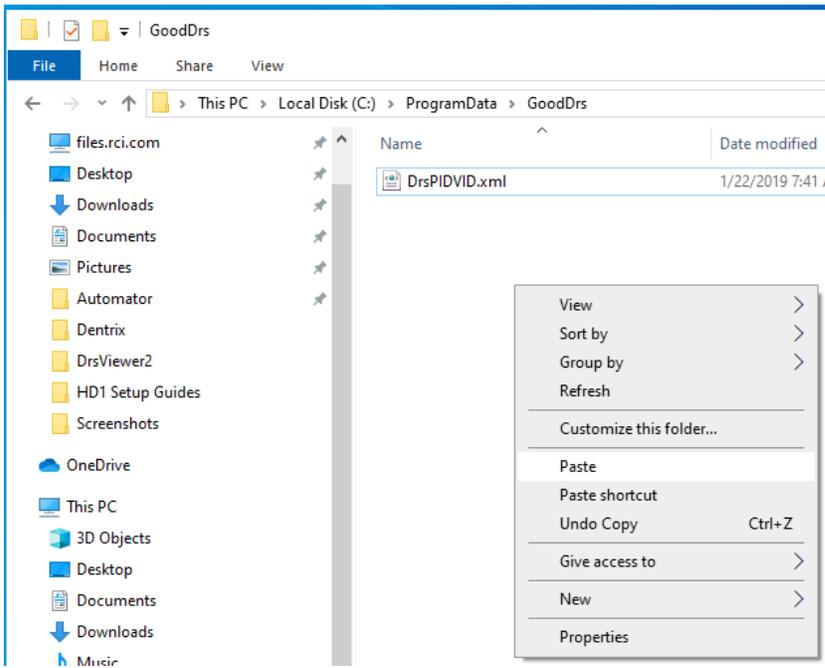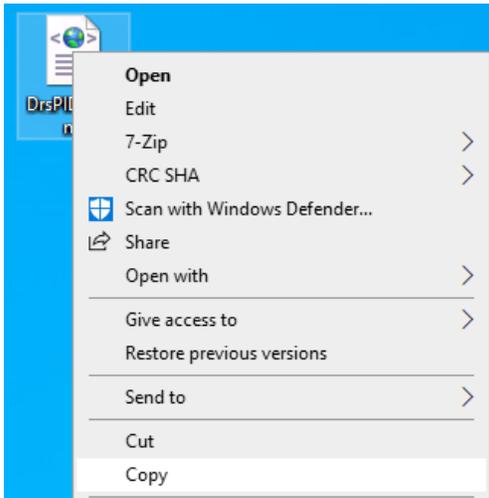


15. Select Replace the file in the destination and Continue

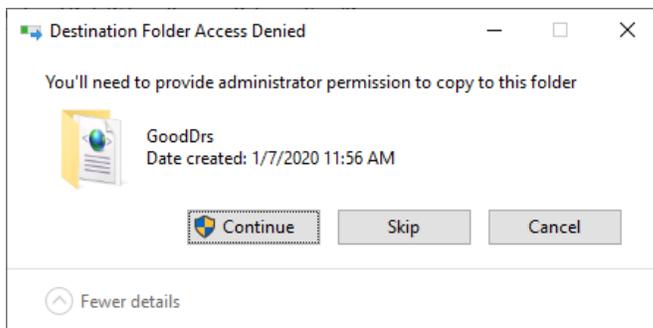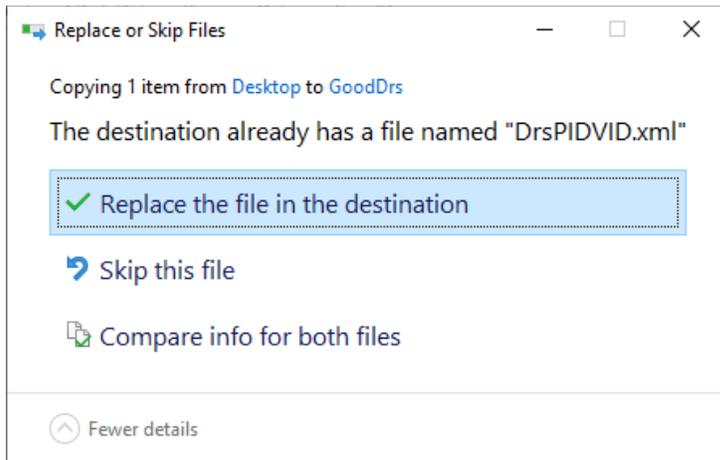


16. Go into the DrsViewer2 application




17. Select Setup > Camera setup

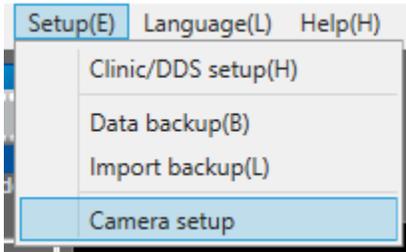


18. Set Resolution to FHD

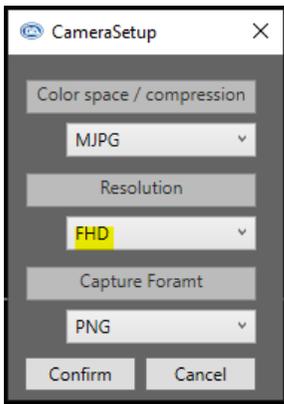

23. Select a patient

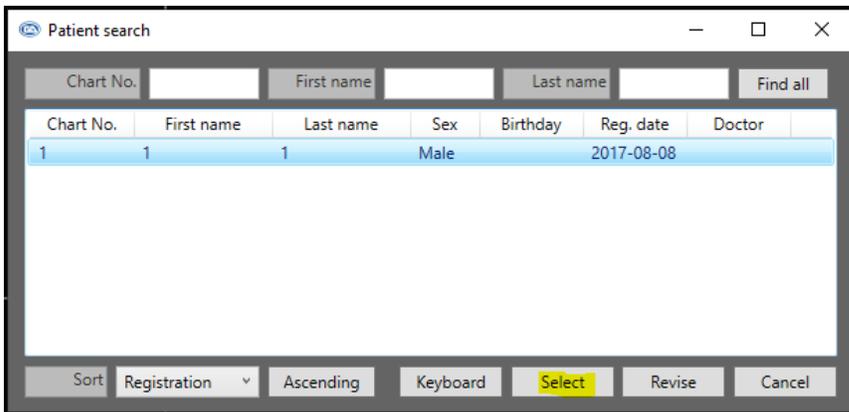

24. Select the Camera button

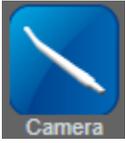

27. A short/quick camera button press, and release will Capture and image from the REALCLOUD HD1 camera.